

TIPS & TRICKS FOR USING YOUR ENVO[®] MASK SAFELY

To ensure you always have the appropriate fit and protection for your respirator, please make sure you are completing a thorough inspection of your envo[®]mask prior to donning.

- **Always inspect your respirator prior to EACH use**

- For maximum protection you need to ensure the filter is completely encased in the mask
- The best way to make sure this happens is to hold the cage that holds the filter in one hand (or on the table) and then place the other cage on top (picture below). Do not hold the filter upside down.
- Please reference the photos below

- **Clean your mask after EVERY use, while wearing gloves used during removal**

- **Your filter is REUSABLE for up to one (1) shift/day** of use unless moist, soiled or torn
 - See: [ENVO Mask Reuse Procedure](#) & [Respirator Service Life by Respirator Type](#)
- Use an approved germicidal wipe to carefully clean the envo[®]mask cage and head gear (be careful not to touch the filter)
- Use an approved germicidal wipe to disinfect your black envo[®]mask storage case (inside and outside)
- Place your envo[®]mask in the black storage case between uses

Envo[®]mask has a great website if you need a video demo on assembling your mask at <https://envomask.com/instructions/>

RECOMMENDED WAY TO ASSEMBLE YOUR MASK



DO NOT HOLD THE FILTER UPSIDE DOWN TO ASSEMBLE



CLOSE UP OF ENCASED FILTER



CORRECT: Note that all “dots” are visible at the mask edge

EXAMPLE OF FILTER NOT FULLY ENCASED



INCORRECT: Note that “dots” near red arrow are NOT visible at the mask edge